


whiteMetal Opaquer

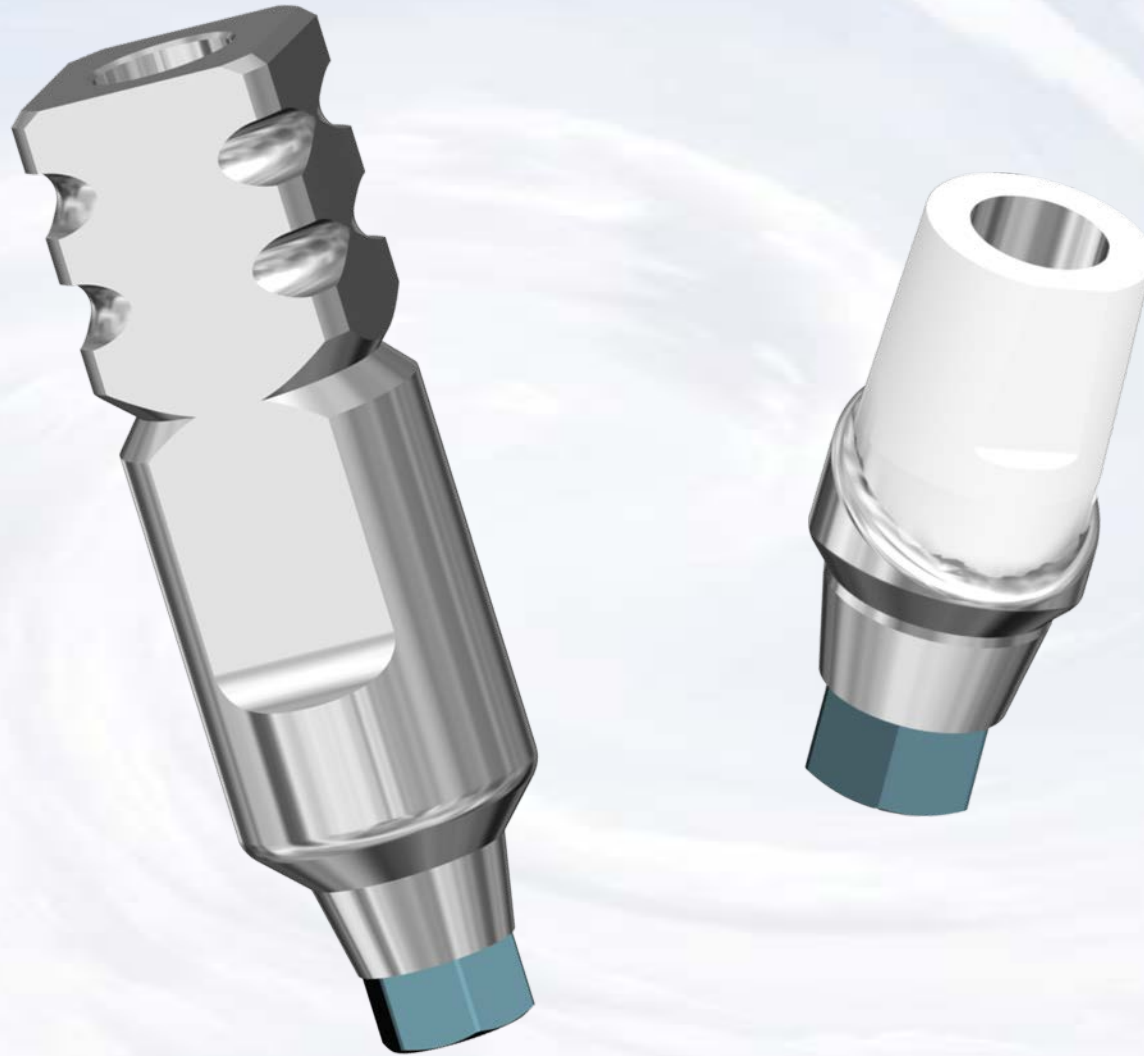

Blue Sky Bio

whiteMetal Opaquer

Prepare metal abutment in a routine fashion



Blue Sky Bio

whiteMetal Opaquer



Sandblast the area of the metal abutment to be covered by whiteMetal. Do not sandblast areas that will not be covered by the final restoration.



Apply one coat of liquid whiteMetal bonding agent to the surface that will be coated with whiteMetal Opaquer.



Blue Sky Bio

whiteMetal Opaquer



Gently air dry the coating of liquid whiteMetal bonding agent. After air drying apply a second coat of whiteMetal liquid bonding agent and air dry again.



Paint a single thin coat of whiteMetal Opaquer on the portion of the abutment that will be covered by the restoration.



Blue Sky Bio

whiteMetal Opaquer



Cure the whiteMetal abutment with a standard curing light



Opaqued abutment on model ready for crown fabrication



Blue Sky Bio

whiteMetal Opaquer



Fabricate

Fabricate ceramic restoration and use resin bonding material for final cementation



Blue Sky Bio